


Quick guide MANO 12

Connections and operation

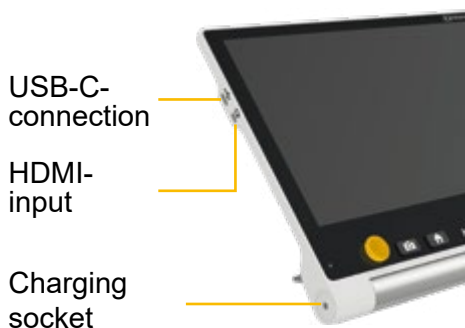

Operation in magnification mode

	move enlarged image
	open main menu
	switch between true colors and semi colors
	long press: back to the true colors

	show freeze image
	save freeze image
	change magnification
	press simultaneously: to switch between line, mask and regular image

Operation in TTS mode

	press and hold: Start TTS mode
--	-----------------------------------

	Capture and read out text
---	---------------------------

Operation in the menu

	select menu item
	exit main menu

	confirm selection
	back

Alternatively, the menu can also be operated via the touchscreen.

Folding mechanism of the integrated stand

Unfold:


press the round button and push down, the stand folds out automatically



Folding:

Take the MANO 12 in both hands and fold the stand together by using light pressure until it clicks into place

press here

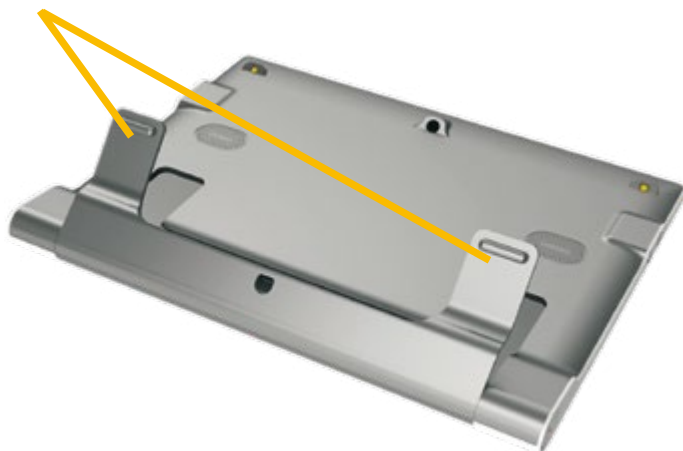

Attention:


To close the stand with as little force as possible, make sure to press on the upper edge of the arms.

Set up

Use with integrated stand


Folding out:

Press the round button

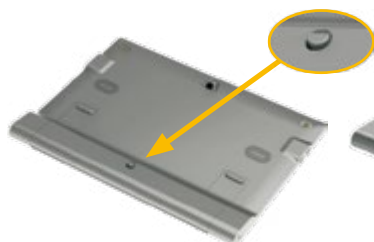


Folding in:

Press the sides of the stand

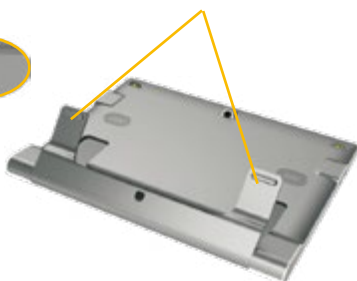

Use with separate reading/writing stand

1

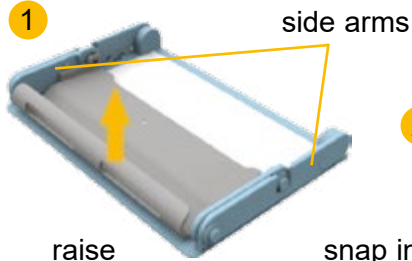

2


snap in



3

fold out


push

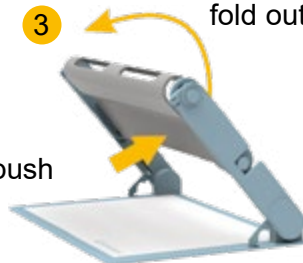

4

fold out

press

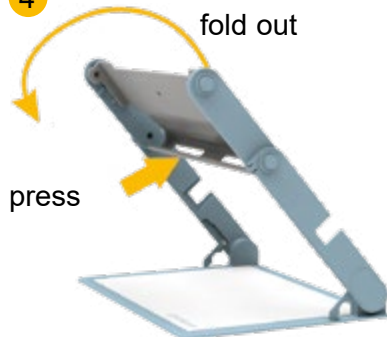

5

insert
MANO 12



6

