
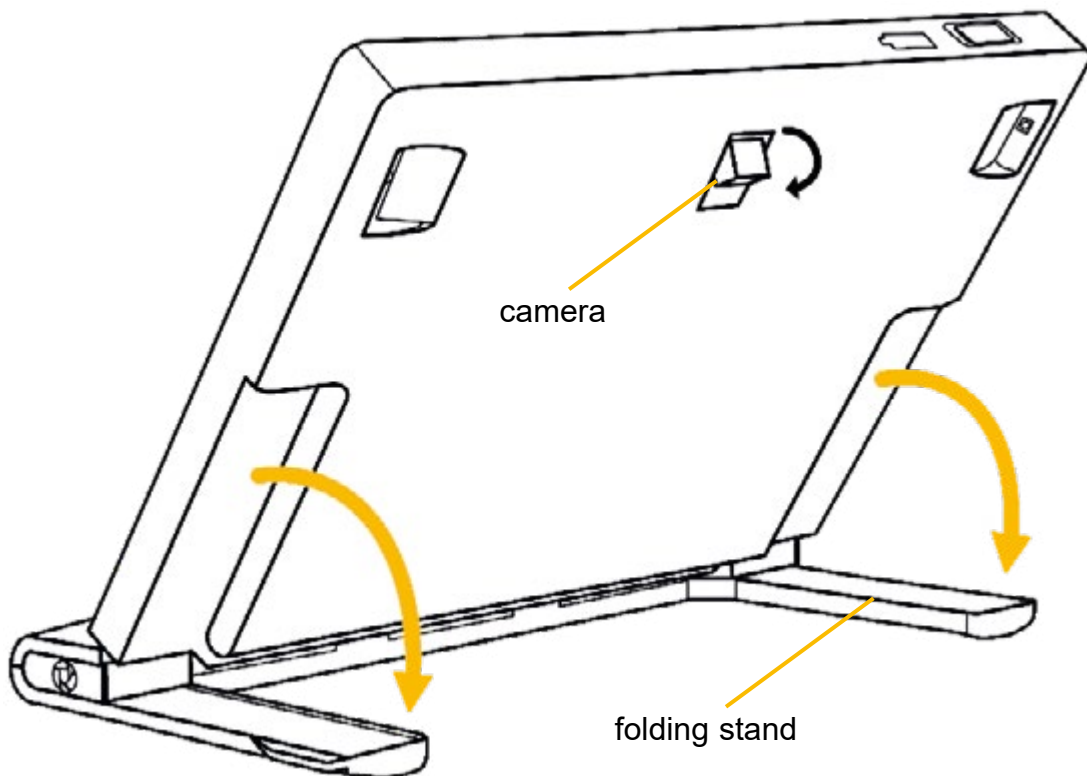


Quick reference guide MANO XXL

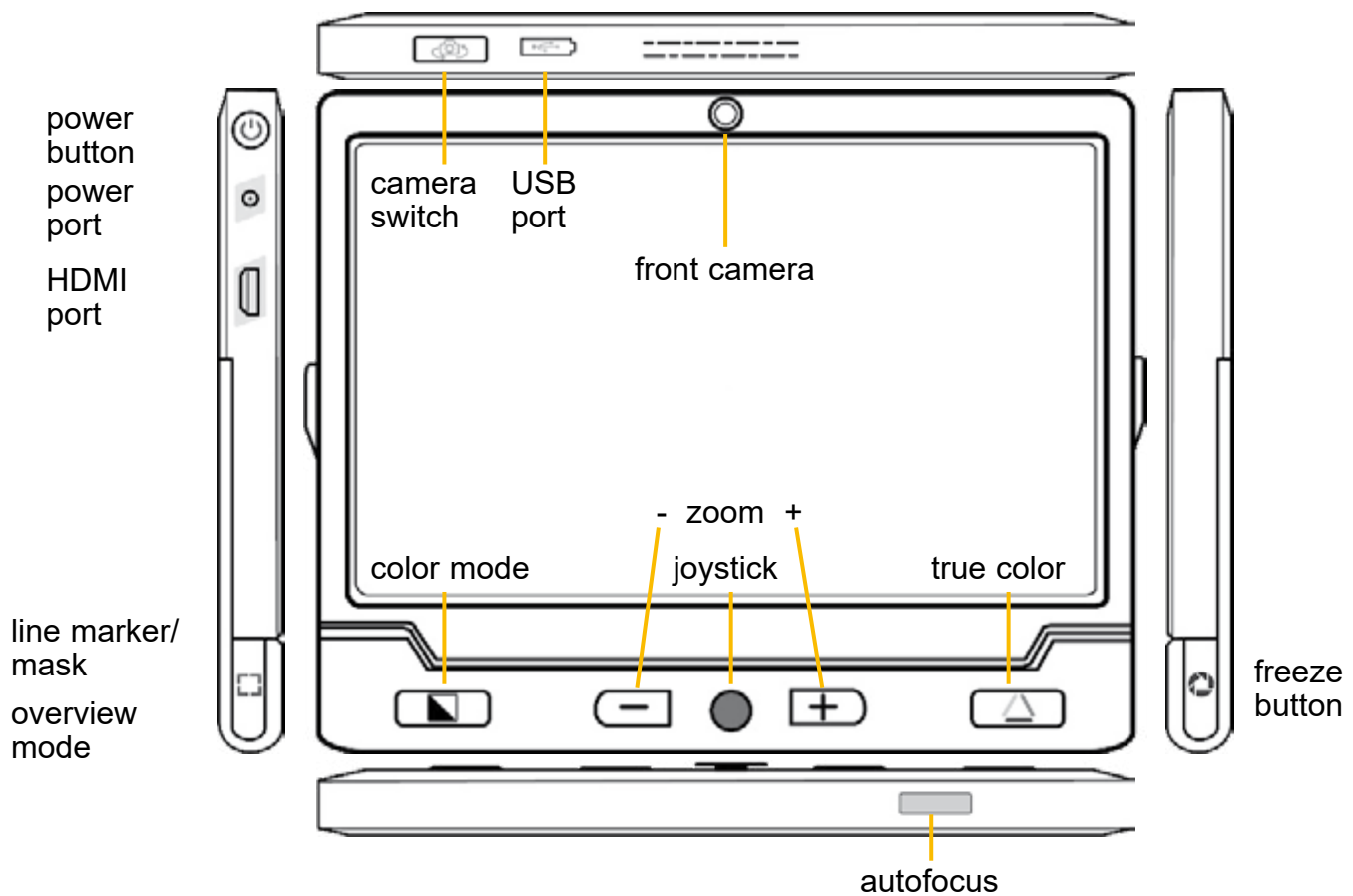

Quick reference guide MANO XXL

Operation			
	short press button: internal menu / status		long press button: switch ON/switch OFF
	short press button: change color mode		long press button: artificial color settings
	short press button: true color mode		long press button: open gallery
	short press button: zoom in in steps		long press button: zoom in continuously
	short press button: zoom out in steps		long press button: zoom out continuously
	short press button: freeze image		long press button: save image
	slide to the left: activate rear camera (distance viewing or reading mode on stand)		slide to the right: activate front camera (mirror function)
	short press button: switch between line marker and mask		long press button: overview mode (when in magnification mode)
	horizontal and vertical navigation in magnification mode; navigation within the picture gallery; adjust size and position of ruler and line masking		
	short press button: autofocus ON/OFF		
image gallery			
	long press button: open/close gallery		 left/right: navigate and select image up/down: next/previous page
	short press button: open selected image		 close image/ back to the gallery
	 + 	in the opened image: go to next/previous image	
	short press: delete selected image press again: confirm deletion		 long press: delete all images press again: confirm deletion
button combinations			
	 + 	increase/decrease screen brightness	
	 + 	increase/decrease contrast (in enhanced color mode)	
		LED lighting ON/OFF	
		switch between: lights all OFF / lights all ON / left light ON, right light OFF / left light OFF, right light ON	