

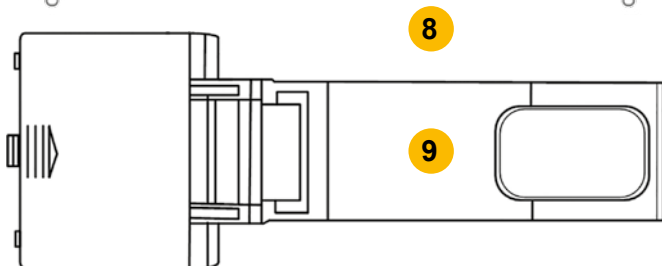
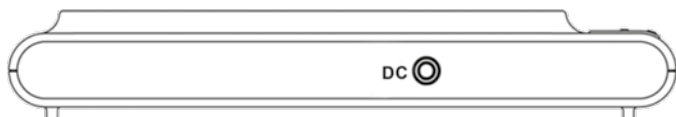


# Quick reference guide

## MANO 5

EN

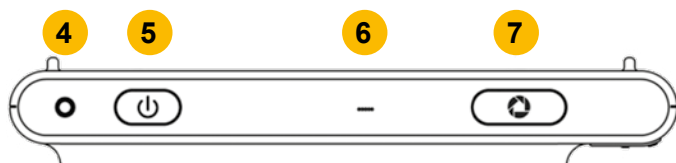

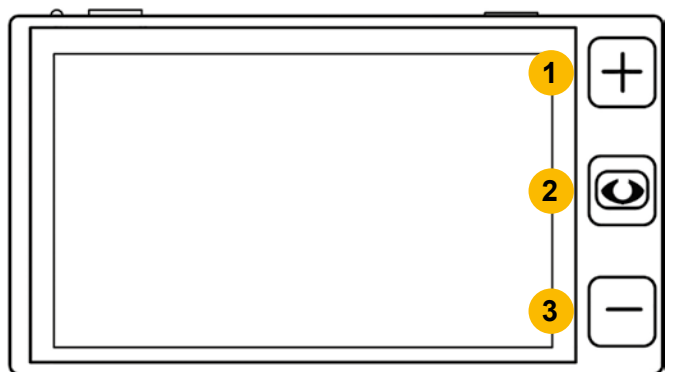
















- 1) zoom in
- 2) color mode
- 3) zoom out
- 4) status LED
- 5) ON/OFF button

- 6) loudspeaker
- 7) freeze image
- 8) power port
- 9) foldable handle

# Quick reference guide

## MANO 5

	short press	long press
	status	switch ON/switch OFF
	change color mode	back to full colors/ change color list
	freeze image	acoustic signals ON/OFF
	zoom in (in steps)	zoom in (continuously)
	zoom out (in steps)	zoom out (continuously)
 + 	LED lighting ON/OFF	
 + 	increase brightness/ move freeze image	
 + 	decrease brightness/ move freeze image	
 + 	freeze image: change direction artificial colors:change contrast	
 + 	long press: energy saving mode ON/OFF	
 + 	long press: reset to factory settings	