


# quick reference guide – MANO EDU XL

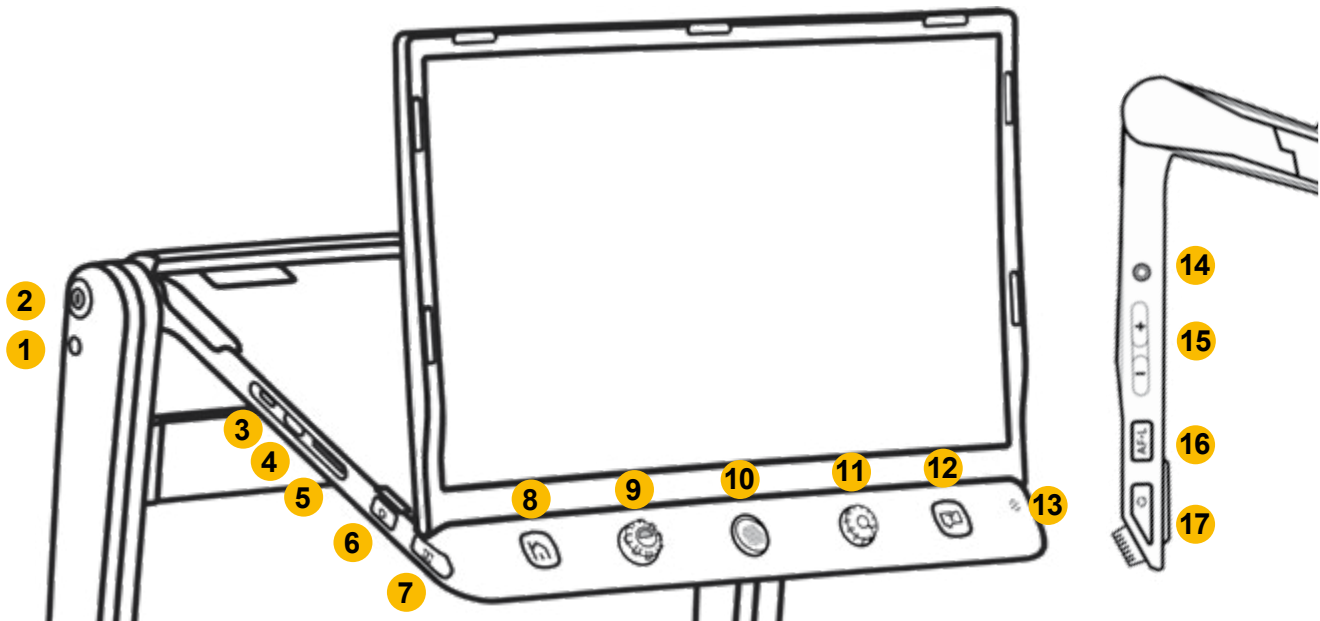

- 1) DC port
- 2) ON/OFF
- 3) HDMI input
- 4) HDMI output
- 5) SD card slot
- 6) camera switch





















- 7) line/mask button
- 8) home
- 9) color mode
- 10) joystick
- 11) zoom
- 12) TTS function

- 13) microphone
- 14) headphone /  
loudspeaker socket
- 15) volume +/-
- 16) autofocus
- 17) freeze frame

- A** Hold the base plate with one hand and unfold the stand
- B** Adjust the display to a comfortable viewing position and optionally raise it at the side and adjust it vertically
- C** Unfold the distance camera and turn it in the desired direction



# quick reference guide – MANO EDU XL

Operation in magnification mode and menus			
	short press: battery status		long press: switch ON/switch OFF/standby
	short press: show/hide menu bar		long press: open menu
	turn controller: change color mode		press controller: switch between true colors and artificial colors
	horizontal and vertical navigation in magnification mode; navigation within the picture gallery; adjust size and position of ruler and line masking		
	turn controller: zoom in/out		press controller: overview mode
	zoom in/out: use the pinch and zoom gesture		
	tap on the magnified image with your fingers and move it		
	short press: switch the camera image long press: enter A4 reading mode		
	short press: switch between line marker and mask		
	short press: freeze image		long press: save image
	short press: start reading (current image section) short press in menus and galleries: confirm selection		long press: start reading (A4 page) long press in galleries: multiple selection
	in menus and galleries: tap with your finger on an icon to select it		
Operation in TTS mode			
	short press: show/hide menu bar long press: exit TTS mode		turn controller: zoom in/out press controller: original zoom
	switch between image- and text mode		text segmentation ON/OFF
	start/stop reading		back/forward by segment/by sentence
	adjust reading speed		in image mode: save image in text mode: save text