


Quick reference guide MANO 6

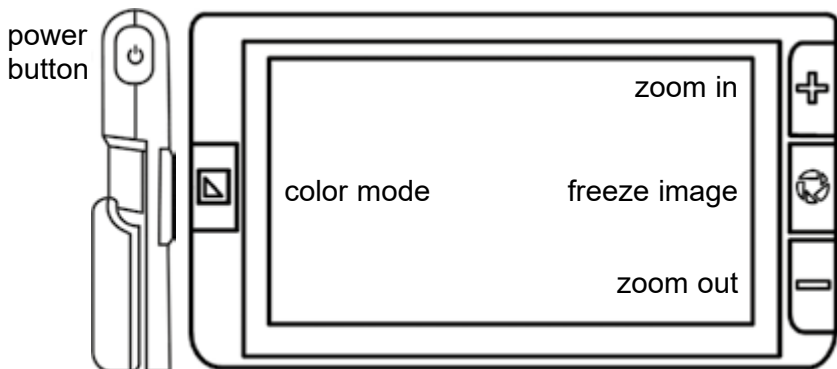







Operation by using but-



	short press button	long press button
	open/close menu	switch ON/switch OFF/ standby mode
	change color mode	artificial color settings
	freeze image	save image
	zoom in in steps	zoom in continuously
	zoom out in steps	zoom out continuously
+ /	short press: increase/decrease screen brightness	
+	short press: LED lighting 50% - 100% - OFF	
+	short press in artificial color mode: switch to true color mode	
	zoom in, zoom out and move the magnified image with multi-touch gestures	

Quick reference guide MANO 6

Touch screen operation

	volume control bar		
	brightness control bar		
	switch between line marker and mask		vibration ON/OFF
	autofocus ON/OFF		energy saving mode ON/OFF
	LED lighting 50% - 100% - OFF		open gallery

Folding stand and handle

power port

